HR Upute za montažu i održavanje ručka brenda BISK S.A.

Čestitamo na kupnji otpornog i kvalitetnog proizvoda.

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GB Fitting and Maintenance Guide for Grip Rails by BISK S.A.

Dear Customer, you have just purchased a durable and good quality product.

„PRO” grip rails by BISK S.A. meet the regulations of harmonized standard PN-EN 12182:2012 and Council Directive 93/42/EEC. These products:

- Carry the CE mark,

- have hygienic approval from the National Institute of Hygiene of the Polish Institute of Public Health,

- are registered with The Office for Registration of Medicinal Products, Medical Devices and Biocidal Products as Class I medical devices. Our grip rails of the „PRO” line have been designed for everybody who wants and needs to feel safe. These products will serve as support while using sanitary ware and performing personal hygiene tasks. This is why we recommend that they be fitted and used in all kinds of personal hygiene rooms. The producer guarantees the stability and static (vertical) capacity of 1,500 N (150 kg) on condition that the product has been fitted correctly. Grip rails have been tested to resist a larger weight (2250 N = 225 kg). Ø 10 mm wall plugs of 50 mm length and Ø 5 mm cross head screws of 60 mm length made of stainless steel are attached with the product, to be used for mounting to the wall. Before starting to fix, please place the mounting plate correctly (drawing 1A, 1B) so that two upper holes are positioned horizontally, then screw the hex head screw M8 (13 mm wrench) tightly*.

ATTENTION: the wall plugs and screws attached are meant to be used in concrete and solid bricks.

The producer does not bear responsibility in case the product is modified in any way or it is mounted in a soft and yielding surface. It is not recommended to drill holes in the rift between tiles. The screws must not be shortened. Should the fixing of the rail arouse any doubt concerning its strength, or should it become unstable and possibly pose hazard to the user's safety, it must not be used.

Maintenance:

Clean with damp soft cloth, preferably soaked with washing-up liquid and wipe dry. Do not use chlorine-, acid- or nitro-solvent-based or abrasive agents.

Guarantee.

The Producer grants a 5-year guarantee. The guarantee does not cover any damages resulting from improper use or damages to the decorative coating caused mechanically or by the application of abrasive or chemically aggressive detergents during product maintenance.

* This does not apply to the middle fixing element of the „L” bar (Ref. No. 04784 and 04785), where the mounting plate should be positioned according to Drawing 1C.

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CZ Návod k montáži a údržbě madel firmy BISK S.A.
Madlá PRO firmy BISK S.A. splňují požadavky harmonizované normy PN-EN 12182:2012 a směrnice Rady 93/42/EEC.