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## GB

### Installation and maintenance instructions for the folding grip rail and fixed grip rail by BISK S.A.

Thank you for choosing the durable quality product.

Folding and fixed grip rails "PRO" by BISK S.A. meet the provisions of harmonised standard PN-EN 12182:2012 and Council Directive 93/42/EEC. The products are attested by the National Institute of Hygiene, bear the CE marking and are registered with the Office for Registration of Medicinal Products, Medical Devices and Biocidal Products as Class 1 Medical Devices.

The "PRO" series grip rails are dedicated for all those who want and need to feel safe. The products are meant to serve as support while using sanitary ware and performing personal hygiene activities. Therefore, they are recommended for fixing and using in all kinds of sanitary and hygiene facilities.

The Producer guarantees stability and static strength (under vertical loading) of 1200 N (120 kg) provided that it is installed properly. The grip rails have been tested under an increased load of 1800 N (180 kg).

The products are wall mounted with the provided  $\phi 10$  mm x 50 wall plugs and  $\phi 6$  x 55 mm round-head screws made of stainless steel.

DOES REFER TO FOLDING GRIP RAIL: Should there be any problems during the installation, the rail should be disassembled by removing the nut of the assembly screw.

NOTE: The plugs and screws attached with the product are meant to be used in concrete and solid bricks.

It is recommended that the product be installed by a qualified professional. Correct, stable and reliable installation guarantees that using the product is safe.

The producer shall not be held responsible if the product was modified in any way, or for the results of fixing in a soft unstable surface.

It is not recommended to drill mounting holes in the joints of two tiles, in the grouting. The mounting plugs or screws must not be shortened. If the grip rail does not seem to be properly mounted or if it has become unstable and might pose a threat to the users, it must not be used.

These products are manufactured and tested with the observance of increased quality and strength standards. They are made of 304 steel according to the AISI Standard. They are finished with a durable lacquer coating (white) or polished steel surface without an extra galvanic coating.

The grip rails are not meant to be used as workout or climbing equipment, or otherwise misused.

Maintenance: Wipe with soft damp cloth, preferably soaked in washing-up liquid, and dry up. Do not apply detergents based on chlorine, acids, nitro diluents, or abrasive agents.

#### Guarantee.

The Producer grants a 5-year guarantee. The guarantee does not cover any damages resulting from improper use or damages to the decorative coating caused mechanically or by the application of abrasive or chemically aggressive detergents during product maintenance.

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### Instrucțiune de montare și întreținere pentru mânerul înclinabil și balustrada fixă a companiei BISK S.A.